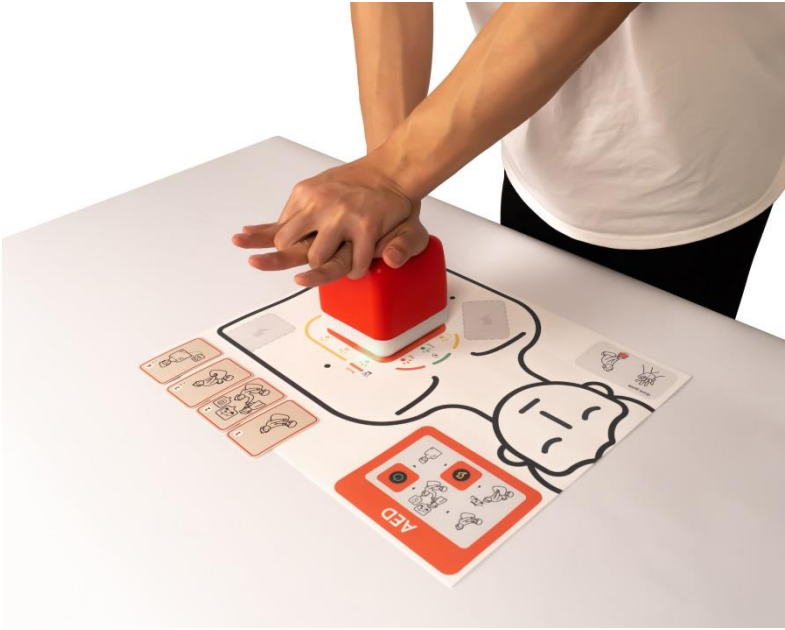


cprCUBE 2 Product Specification

Name	cprCUBE 2, chest compression CPR training feedback device		
Item no.	CC2G	Manufacturer / Country of Origin	I.M.LAB Inc. / South Korea
Picture			
Use	Chest compression CPR training device equipped with audiovisual real-time feedback through sound and LED.		
Features	● Polyurethane foam to simulate the actual human chest density ● Visual feedback for compression depth (Less than 5cm: Yellow LED light, Equal to or more than 5cm: Green LED light) ● Visual feedback for compression rate (Less than 100 times per minute: Yellow LED light, 100 times to 120 times per minute (proper rate): Green LED light, More than 120 times per minute: Red LED light) ● If recoil is not properly done (less than 1cm after the compression), visual feedback through LED (red LED light) ● For continuous correct compression (correct depth, rate and recoil) of 10, 20 and 30 times, distinct sound feedback provided to notify the users of CPR performance level		

	● Compression, rate and recoil feedback guide on the guide board and scenario provided for practicing each step of chain of survival ● Total of 4 scenario cards for learning CPR steps (Patient recognition -> Call for help -> Check breath and pulse -> CPR) and flip the cards to use them as AED pads to simulate AED operation ● Using AED trainer picture and scenario cards for role-playing training ● Requires 4 “AA” alkaline batteries (not included) ● Sound guide (metronome sound) for practicing proper compression rate ● Power button is located at the bottom of the device and automatically turns off if there is no activity for more than 10 minutes, to minimize the power loss
Specification	● Dimensions: - Package box: 105mm X 105mm X 130mm - cprCUBE 2 (device): 100mm X 100mm X 110mm - Board: 400mm X 330mm X 0.5mm ● Weight: 240g (cprCUBE 2 without batteries) ● Power source: DC 6V, 4 AA batteries
Components	1. cprCUBE 2 1 ea 2. Guide board 1 ea 3. User manual 1ea 4. Scenario card 1set (4ea)